


COVID-19 & Influenza A+B Antigen Combo Rapid Test For Self-Testing

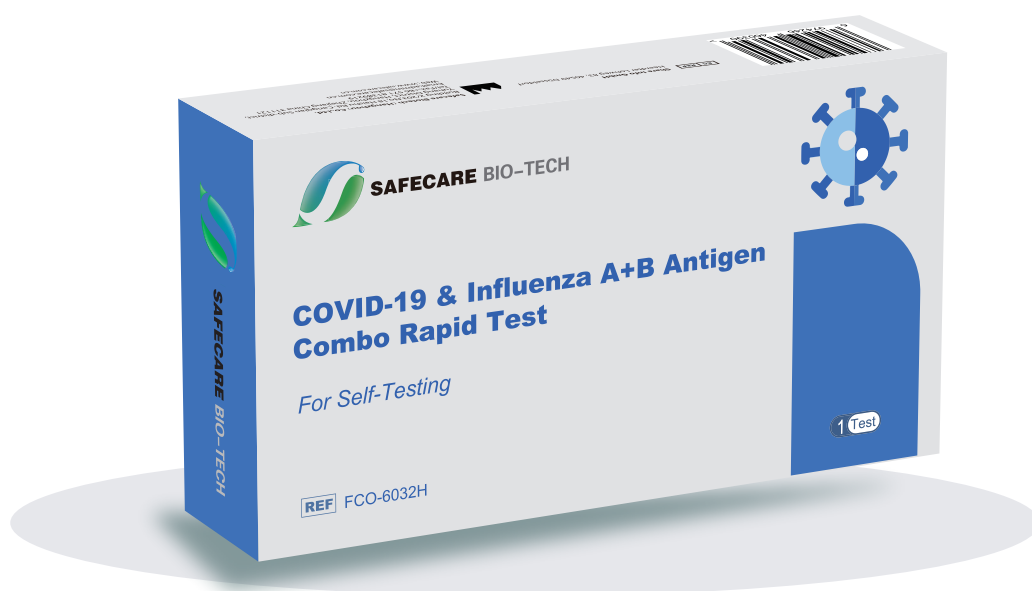

Intended Use:

The COVID-19 & Influenza A+B Antigen Combo Rapid Test is a single-use test kit intended for qualitative detection of nucleocapsid protein antigen of influenza A and B viral antigens and COVID-19 Antigen from nasal swab specimens at home.

COVID-19 & Influenza A+B Antigen Combo Rapid Test For Self-Testing

Product Features

- ◇ *High Accuracy*
- ◇ *Fast Results in 15 mins*
- ◇ *Easy Specimen Collection*
- ◇ *Easy for Private Home Use*
- ◇ *Simple operation, No equipment required*




Contents

- ◇ *COVID-19 & Influenza A+B Antigen combo Test-1*
- ◇ *Extraction tube with buffer-1*
- ◇ *Sterilized nasal swab-1*
- ◇ *Package insert-1*
- ◇ *Waste bag-1*




使用过程


Convenient Procedure

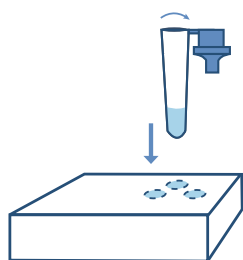


Peel off the foil film on the extraction tube and insert into the hole

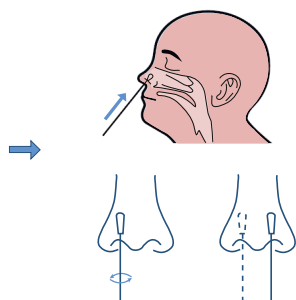


Insert nasal swab to an appropriate depth, roll 5 times in each nostril

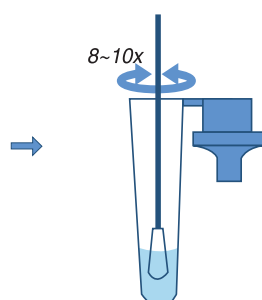


Rotate the swab 8-10 times

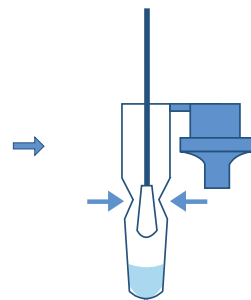


Press the swab and release as much liquid as possible

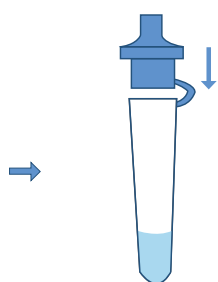


Place the cap

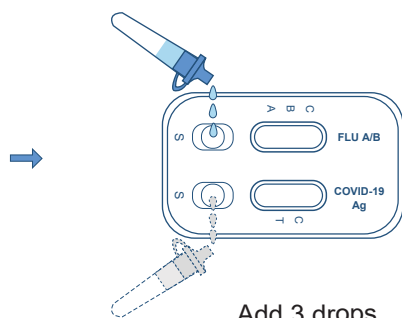

Add 3 drops to each sample well(S)

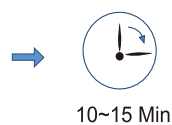

10~15 Min

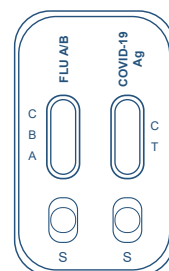

Read the result

Results Interpretation

For COVID-19



Positive



Negative



Invalid

For FLU A/B



Positive



Positive



Positive




Negative

Invalid

包装信息

Packing Information

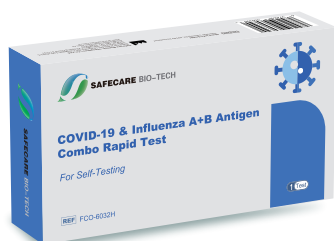

Brand:SAFECARE

Product name:COVID-19 & Influenza A+B Antigen Combo Rapid Test For Self-Testing

Package:1T/box

CTN Size:63×37×30cm

QTY:320Tests/CTN

G.W.:9.8 kgs

N.W.:7kgs

Inner box size:122×67×22mm